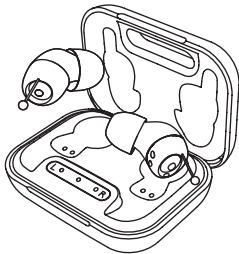
Hearizōn

ES Manual de instrucciones

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EN User guide

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ÍNDICE

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INDEX

Important Safety Information	29
Box Contents	30
Parts Overview	30
Sound Amplifiers	30
Charging Case	31
Getting Started	31
Turning the device On	30
Selecting the Right Ear Tip	32
Powering On/Off.....	33
How to Wear the Devices	34
Adjusting the Volume	35
Charging the Devices	35
Charging the Amplifiers	35
Charging the Case	36
Cleaning and Maintenance.....	37
Troubleshooting	37
Amplifier Issues	37
Charging Case Issues	38
Technical Specifications	39

IMPORTANT SAFETY INFORMATION

- Read all instructions in this user manual before using the product.
- Remove the devices from your ears if you experience pain or discomfort when inserting or wearing them.
- Your brain will need time, practice, and patience to adjust to the new sounds provided by HEARIZON.
- Avoid dropping or striking the product.
- Do not submerge the devices in any liquid. Remove them before swimming or bathing, and allow your ear canals to dry completely before reinserting them.
- Daily maintenance is required to keep the amplifiers clean. Always make sure the ear tips are free of moisture or debris.
- The external surfaces of the device should be kept clean. Earwax buildup on the ear tips must be cleaned daily using the cleaning tool included in the box.
- Charge the devices regularly to avoid battery damage. The recommended charging interval is less than 28 days.
- Do not expose the amplifiers to magnets or magnetized objects.
- Keep the devices out of reach of children and pets.
- Store the product in environments free from excessive heat or humidity.
- Do not insert sharp objects into the microphone to prevent damage.

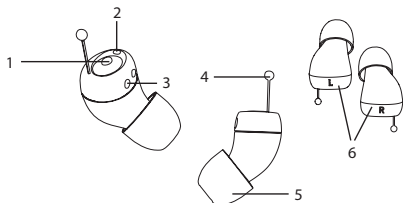
BOX CONTENTS

- 2x Sound Amplifiers (1x right, 1x left)
- 1x Charging Case
- 3x Pairs of Ear Tips (Sizes S, M, L)
- 1x USB Cable
- 1x Cleaning Tool
- 1x User Manual

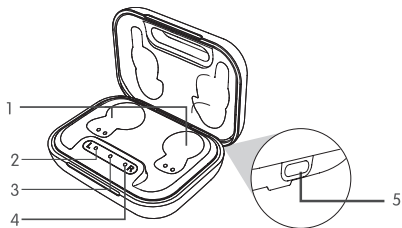
PARTS OVERVIEW

SOUND AMPLIFIERS

1. Multi-function Button: Power On/Off and Volume Control
2. Microphone
3. Charging Contacts
4. Pull Tab
5. Ear Tip
6. Left (L) and Right (R) Markings

CHARGING CASE

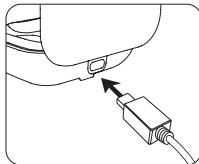
1. Magnetic Charging Slots
2. Left Amplifier Charging Indicator (L)
3. Charging Case Battery Level Indicator
4. Right Amplifier Charging Indicator (R)
5. Charging Port

GETTING STARTED

Turning the Device On

- Plug the charging case into a power outlet using the provided cable.
- Remove the protective film from the amplifiers to ensure proper contact with the charging case.
- The case light will turn orange while charging. Once fully charged, the light will turn green.

SELECTING THE RIGHT EAR TIP

- **Step 1:** Choose the size that fits you best (S, M, or L). It should fit snugly in your ear without feeling loose.

L

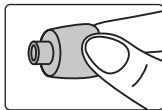
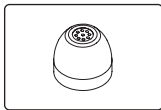
M

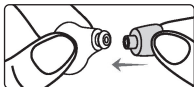
S

- **Step 2:** Turn the ear tip inside out so the attachment point is exposed.

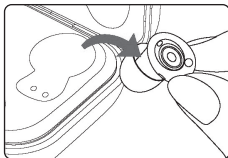
- **Step 3:** Hold the earbud with one hand and the ear tip with the other. Press firmly to ensure the tip is securely attached, then flip it back over so it fully covers the earbud.

POWERING ON/OFF

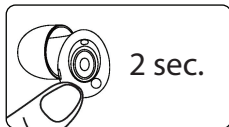
There are two ways to turn the devices on and off:

Powering On

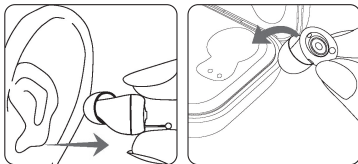
Method 1: The devices will automatically power on approximately 12 seconds after being removed from the charging case.

Method 2: If the devices are off, press and hold the multi-function button for 2 seconds, then wait about 15 seconds for them to start.

Powering Off

Method 1: The devices will automatically power off and begin charging when placed correctly into the charging case.

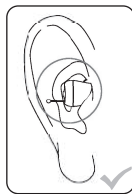
Method 2: When the devices are on, press and hold the multi-function button for 2 seconds to turn them off.

HOW TO WEAR THE DEVICES

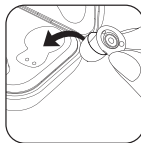
Inserting

- Insert each sound amplifier into your ear canal with the pull tab facing downward.
- The tab should be positioned as shown in the illustration.

Removing

- Gently pull on the tab to remove the device from your ear.
- Place each sound amplifier into its corresponding slot in the charging case.

Note: The sound amplifiers will emit a whistling sound if not inserted correctly. Remove the affected amplifier and reinsert it properly. If the whistling persists, try using a larger ear tip and insert it again. If the problem continues, please refer to the *Troubleshooting* section for further assistance.

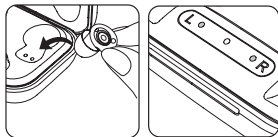
VOLUME

- The sound amplifiers have 5 volume levels.
- Press the multi-function button once to adjust the volume.
After reaching the maximum level, pressing the button again will reset it to level 1.
- A tone ("Di") will confirm that the maximum volume has been reached.

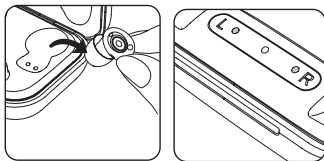
CHARGING CASE

Charging the Sound Amplifiers

The devices begin charging automatically when placed in the correct slot of the charging case.

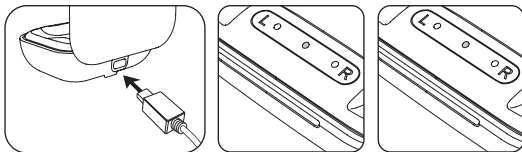
- Orange lights indicate that the devices are charging — both the right unit (R indicator) and the left unit (L indicator).

- When the amplifiers are fully charged, the indicator light will turn green — either on the right side (R), the left side (L), or both. (The lights will turn off automatically after a few minutes.)

CHARGING THE CASE

- Plug the charging case into the charging cable and connect it to a power outlet.
- The center indicator light will turn orange, indicating that the case is charging. Once fully charged, the light will turn green and will turn off automatically after a few seconds.

Note: If the battery level drops below 30%, the center indicator will flash orange.

CLEANING AND MAINTENANCE

- Use the cleaning tool included in the box. Remove the ear tips from the devices and use the tool to eliminate earwax, dirt, or any debris from both the ear tips and the amplifiers themselves.
- Keep the charging case clean and ensure that the connectors are free of debris to allow proper charging of the devices.

TROUBLESHOOTING

Devices

ISSUES	POSSIBLE CAUSE	RECOMMENDED SOLUTION
Feedback or "whistling"	Devices are not properly inserted into the ears.	Remove and reinsert following the instructions.
	Volume is too high.	Lower the volume.
	Ear tips are damaged or blocked.	Check the ear tips and replace them if necessary.
	Holding an object too close to the sound amplifiers.	Move your hand or object away to create more space between the amplifiers and objects.
	Excess earwax in the ear canal.	Clean your ears and consult your doctor if necessary.

No sound	The button press duration was too short to power on the devices.	Power on the sound amplifiers correctly.
	Battery is empty.	Charge the devices.
	Microphone is blocked.	Clean any debris or earwax from the microphone.
Weak or muffled sound	Battery is low.	Charge the devices.
	Volume is too low.	Increase the volume.
	Microphone is blocked.	Clean any debris or earwax from the microphone.

CHARGING CASE

ISSUES	POSSIBLE CAUSE	RECOMMENDED SOLUTION
Charging indicator does not light up or blink	Power adapter is not receiving electricity.	Connect or reconnect the adapter to the power outlet.
	Charging case is not properly connected to the USB cable.	Check and reconnect the case to the USB cable securely.
Case does not charge the sound amplifiers	Sound amplifiers are not correctly placed in their charging slots.	Insert the amplifiers into the correct slots in the case to begin charging.

ISSUES	POSSIBLE CAUSE	RECOMMENDED SOLUTION
Charging case cannot charge the amplifiers	Charging contacts are dirty.	Clean the charging contacts on both the amplifiers and the charging case.

TECHNICAL SPECIFICATIONS

Model	JH-A490
Maximum Output (OSPL@90)	$\leq 108+3\text{dB}$
HFA-OSPL@90	$105+4\text{dB}$
Maximum Full-Gain Output	$\leq 31+3\text{dB}$
Full-Gain Value (HFA FOG@50)	$28+5\text{dB}$
Total Harmonic Distortion (THD)	$\leq 2-3\%$
Equivalent Input Noise (EIN)	$\leq 29+3\text{ dB}$
Frequency Range (f1-f2)	290 Hz-4500 Hz
Operating Conditions	Temperature: $0\text{ }^{\circ}\text{C} \sim 40\text{ }^{\circ}\text{C}$
	Relative Humidity: 30 %-75 % RH
	Atmospheric Pressure Range: 86 – 106 kPa

Storage and Transport	Temperature: -10 °C~55 °C
	Relative Humidity: 10% – 90% RH
	Atmospheric Pressure Range: 70 – 106 kPa
Battery Life	≥14 h

ELECTRICAL DEVICES

The crossed-out wheeled bin symbol indicates that this product must not be disposed of with regular household waste. Dispose of electrical or electronic equipment through your local collection and recycling system. For more information about proper disposal and recycling, contact your local waste disposal authority, the store where you purchased the product, or the manufacturer.

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